ESTESIL H₂TOP[®] REGULAR, HIGH AND EXTRA HIGH FLOWABILITY

Are addition reaction impression materials in polyvinylsiloxane with highly wettable surfactant hydrophilic fillers used as precision correctors. Available in different formulations to meet the requirements of the main impression taking technique. They are supplied in the cartridge version to be used with mixing guns in a 1:1 ratio.

Using the Gun Mix 1:1 cartridge proceed in the following way:

1. Insert the cartridge into the mixing gun.
2. Activate the cartridge removing the protection cap.
3. Insert the mixing cannula on the cartridge.
4. Dispense about 2/3 cm. of material, and eliminate it, before placing it in place directly with the special tips, or injecting it with special syringes used to achieve a precise application into the area where a more detailed impression taking is necessary.
5. Do not remove the mixing cannula once it has been used (it acts as a protection plug).

GENERAL SAFETY INFORMATION

The formulations of the impression materials **ESTESIL H₂TOP[®]** contain curing monomers and surfactants which may irritate the skin, the eyes and the oral mucosa. They may cause allergic contact dermatitis in particularly susceptible subjects.

CONTRAINDICATIONS

1. Avoid contact with the eyes to prevent potential irritations and /or accidental damage to the cornea. In case of accidental contact with the eyes rinse with plenty of water and call a doctor.
2. Avoid contact with the skin to prevent potential irritations or cause any allergic reactions (in susceptible subjects). In case of contact with the skin remove the material and rinse with plenty of water and neutral soap. In case of sensitization or reddish rash, stop using the material and seek medical assistance.
3. Do not ingest the product. In case of accidental ingestion drink plenty of water. In general the material is not dangerous if swallowed in small quantities. Seek medical advice in case of symptoms of a different nature.

PRECAUTIONS

1. **ESTESIL H₂TOP[®]** is intended only for dentistry, following the specific "instructions for use". Any different and/or imprudent use or and/or use not compliant with the specifications expressly provided is to be considered the dentist's sole discretion and responsibility.
2. Do not use dental materials containing hydrogen peroxide in combination with **ESTESIL H₂TOP[®]** because they may prevent its correct hardening.
3. Thanks to its high thixotropy and formulation, the material should be dispensed easily. DO NOT apply too much pressure to avoid affecting the extrusion of the material or breaking the packaging.

SUGGESTIONS FOR THE USE OF ESTESIL H₂TOP[®] HEAVY, REGULAR, HIGH AND EXTRA HIGH FLOWABILITY

TYPE	COLOR	CODE	SUGGESTIONS FOR USE
PUTTY SOFT <i>fast</i>	Indigo Blue	B	Base for dual phase impression with two viscosities (putty-wash technique)
RF Regular Flowability <i>fast</i>	Orange	R	B+R: precision impression for supragingival preparations when you need closer viscosities
HF High Flowability <i>fast</i>	Coral	F	B+F or Δ+F or E+F: precision impression for supragingival or luxta gingival preparations
EHF Extra High Flowability <i>fast</i>	Red Grapes	H	B+H o Δ+H: precision impression for subgingival reading

ESTESIL H₂TOP[®] REGULAR, HIGH ET EXTRA HIGH FLOWABILITY

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ESTESIL H₂TOP[®] REGULAR, HIGH E EXTRA HIGH FLOWABILITY

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Tokuyama Dental Italy S.r.l.