Resin-based Dental Restorative Material

ESTELITE UNIVERSAL FLOW SYRINGE**ESTELITE UNIVERSAL FLOW** PLT

CE 1639

UK
0120
0120

Instructions for Use

Tokuyama Dental Corporation

38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan

Tel: +81-3-3835-7201

URL: <http://www.tokuyama-dental.com/>

| | | | | | | | |
|--------------------|-----|-------------------|-----|-------------------|-----|-----------------|-----|
| ENGLISH | 2 | ČESKY | 10 | DANSK | 19 | DEUTSCH | 28 |
| EESTI KEEL | 37 | FRANÇAIS | 46 | ITALIANO | 56 | LATVISKI | 66 |
| LIETUVIŲ K. | 75 | MAGYAR | 83 | NEDERLANDS | 92 | NORSK | 101 |
| POLSKI | 110 | SLOVENČINA | 119 | SUOMI | 128 | SVENSKA | 137 |

This product is certified as a medical device under the legislation of the European Union on medical devices by SGS CE1639, exclusively for the indication(s) shown in this IFU. Other non-medical uses ascribed to this product are not within the scope of CE certification, and users should be aware product performance and/or safety has not been evaluated by SGS for those purposes.

Carefully read all information, precautions, notes and materials before using ESTELITE UNIVERSAL FLOW.

PRODUCT DESCRIPTION AND GENERAL INFORMATION

- 1) ESTELITE UNIVERSAL FLOW is a low viscosity, light-cured, radiopaque composite resin for use in anterior and posterior restorations. Direct placement of ESTELITE UNIVERSAL FLOW into a prepared cavity allows for easy handling. It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 1 and Class 2 (Group 1) material by ISO 4049.
- 2) ESTELITE UNIVERSAL FLOW incorporates Radical-Amplified Photopolymerization initiator technology (RAP technology). RAP technology facilitates a shortened light curing time and ample working time. Please see the table depicting the relationship between curing time and increment depth

INDICATIONS FOR FILLING AND CURING

- 3) ESTELITE UNIVERSAL FLOW comes in three viscosities, High, Medium and Super Low. Each type contains spherical silica-zirconia filler and composite filler as indicated in below table. The composite filler offers reduced polymerization shrinkage stress due to its round shape. The spherical filler contained in ESTELITE UNIVERSAL FLOW (mean particle size: 200 nm, particle size range: 100 to 300 nm) facilitates excellent gloss retention and wear resistance.

| Type | Filler Loading |
|-----------|-------------------------------|
| High | 69% by weight (55% by volume) |
| Medium | 71% by weight (57% by volume) |
| Super Low | 70% by weight (56% by volume) |

- 4) ESTELITE UNIVERSAL FLOW contains Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), Bisphenol A polyethoxy methacrylate (Bis-MPEPP), Triethylene glycol dimethacrylate (TEGDMA), 1,6-bis(methacryloxyethylcarbonylamino)trimethyl hexane (UDMA), Mequinol, Dibutyl hydroxyl

toluene and UV absorber.

5) ESTELITE UNIVERSAL FLOW is available in either SYRINGE (L-SYRINGE or SYRINGE (Medium only)) or Pre Loaded Tip (PLT (Medium only)).

■ SHADE

Available shade in each type is as below.

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- Opalescent shades (OPA2, OPA3, OPA4) have adequate opacity designed for blocking out the dark shine that comes through the oral cavity (Class III and IV restoration). The opalescent shades also are designed for masking slight stain or reconstructing a highly opaque tooth.

- Please check with your local distributor for available shades.

■ INDICATIONS

- Direct anterior and posterior restorations
- Cavity base or liner
- Blocking out cavity undercuts before fabricating indirect restorations
- Repair of porcelain/composite

■ CONTRAINDICATIONS

ESTELITE UNIVERSAL FLOW contains methacrylic monomers and UV absorber. DO NOT use ESTELITE UNIVERSAL FLOW for patients allergic to or hypersensitive to methacrylic and related monomers, to UV absorber or to any of the other ingredients.

■ PRECAUTIONS

- 1) DO NOT use ESTELITE UNIVERSAL FLOW for any purpose other than those listed in these instructions. Use ESTELITE UNIVERSAL FLOW only as directed herein.

- 2) ESTELITE UNIVERSAL FLOW is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor use by non-dental care professionals.
- 3) DO NOT use ESTELITE UNIVERSAL FLOW if the safety seals are broken or appear to have been tampered with.
- 4) If ESTELITE UNIVERSAL FLOW causes an allergic reaction or hypersensitivity, discontinue its use immediately.
- 5) Use examination gloves (plastic, vinyl or latex) at all times when handling ESTELITE UNIVERSAL FLOW to avoid the possibility of allergic reactions from methacrylic monomers or UV absorber. Note: Certain substance/materials may penetrate through examination gloves. If ESTELITE UNIVERSAL FLOW comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- 6) Avoid contact of ESTELITE UNIVERSAL FLOW with eyes, mucosal membrane, skin and clothing.
 - If ESTELITE UNIVERSAL FLOW comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If ESTELITE UNIVERSAL FLOW comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is completed.
 - If ESTELITE UNIVERSAL FLOW comes into contact with the skin or clothing, immediately saturate the area with an alcohol soaked cotton swab or gauze.
 - Instruct the patient to rinse his mouth immediately after treatment.
- 7) ESTELITE UNIVERSAL FLOW should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- 8) To avoid the unintentional ingestion of ESTELITE UNIVERSAL FLOW, do not leave it unsupervised within the reach of patients and children.
- 9) Clean the placement instruments and brushes with alcohol after use.
- 10) When using a light-curing unit, protective eye shields, glasses or goggles should be worn at all times.
- 11) This product is designed to be used at room temperature (18 - 30°C / 62 - 84°F). Allow product to reach room temperature prior to use. Cold material may be

difficult to extrude.

INDICATIONS FOR FILLING AND CURING

ESTELITE UNIVERSAL FLOW is designed to be cured by either a halogen, LED, or Plasma Arc light with a wavelength of 400-500 nm. Be sure to light-cure ESTELITE UNIVERSAL FLOW extra-orally and check the time needed for complete hardening of ESTELITE UNIVERSAL FLOW with your light-curing unit before performing the bonding procedure. The following table summarizes the relationship between curing time and increment depth.

Relationship between curing time and increment depth:

| Light type | Intensity (mW/cm ²) | Curing time (seconds) | Increment depth (mm) ¹⁾ | | |
|------------|---------------------------------|-----------------------|------------------------------------|-----|----------------------|
| | | | CE, BW, A1, A2, A3, A3.5, A4, B3 | A5 | OPA2 OPA3 OPA4 |
| Halogen | 600 | 10 | 1.8 | 1.6 | 1.3 |
| | | 20 | 2.3 | 2.0 | 1.6 |
| | 800 | 10 | 2.0 | 1.7 | 1.5 |
| | | 20 | 2.4 | 2.1 | 1.7 |
| | 1200 | 5 | 1.8 | 1.6 | 1.3 |
| | | 10 | 2.2 | 2.0 | 1.6 |
| LED | 600 | 10 | 1.7 | 1.5 | 1.2 |
| | | 20 | 2.1 | 1.8 | 1.4 |
| | 800 | 10 | 1.9 | 1.7 | 1.3 |
| | | 20 | 2.2 | 2.0 | 1.6 |
| | 1200 | 10 | 2.0 | 1.8 | 1.5 |
| | 2300 | 3 | 1.7 | 1.5 | 1.2 |
| Plasma Arc | 1200 | 3 | 1.7 | 1.5 | 1.1 |

- 1) Increment depth was determined on the basis of test results performed in accordance with “depth of cure” of ISO 4049.

■ SPECIAL NOTES FOR THE USE OF PLT

- 1) PLTs are designed for single patient use only. To avoid cross infection, Do not re-cap and/or re-use the PLT tip once material has been dispensed for that patient.
- 2) Dispensers are not provided for ESTELITE UNIVERSAL FLOW. Use a dispenser that fits the PLT of ESTELITE UNIVERSAL FLOW. For proper use and proper disinfection, see manufacturer’s instructions.
- 3) Use light, controlled pressure to prevent any continued extrusion of material following pressure release.

■ STORAGE

- 1) Store ESTELITE UNIVERSAL FLOW at temperatures between 0 - 25°C (32 - 77 °F).
- 2) AVOID direct exposure to light and heat.
- 3) DO NOT use ESTELITE UNIVERSAL FLOW after the indicated date of expiration on the syringe or PLT package.

■ DISPOSAL

To safely dispose of excess ESTELITE UNIVERSAL FLOW, extrude unused portion from SYRINGE or PLT and light-cure before disposal.

Follow local instructions for disposal.

Refer to the packaging waste disposal guide shown on the end of this IFU.

■ CLINICAL PROCEDURE

1. Cleaning

Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.

2. Shade Selection

Prior to isolation of tooth, select the appropriate shade of ESTELITE UNIVERSAL FLOW using shade guide provided for ESTELITE UNIVERSAL FLOW.

3. Isolation

A rubber dam is the preferred method of isolation.

4. Cavity Preparation

Prepare the cavity. In case where no cavity preparation has been made (caries-free cervical defects), clean the tooth surface with a rubber cup and a fluoride-free cleaning paste. Rinse thoroughly with water.

- In the case of porcelain/composite repairs, roughen the surface with a bur or a diamond point to prepare the area for adhesion; apply phosphoric acid etch for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling agent according to manufacturer's instructions.

5. Pulp Protection

Calcium hydroxide should be applied if the cavity is in close proximity to the pulp. **DO NOT USE EUGENOL-BASED MATERIALS** to protect the pulp as these materials will inhibit curing ESTELITE UNIVERSAL FLOW. ESTELITE UNIVERSAL FLOW is not indicated for direct pulp capping.

6. Bonding System

Apply a bonding system according to its manufacturer's instructions.

- In case of porcelain/composite repairs be sure to condition the surface of porcelain/composite with a silane coupling agent **BEFORE** performing the bonding procedure.

7. Dispensing

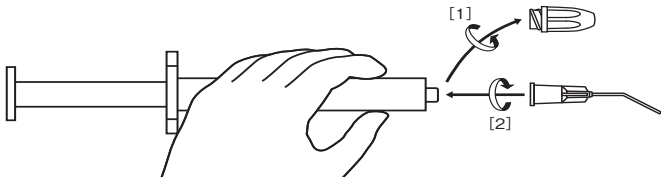
7-1. PLT (Medium)

Please read **■ SPECIAL NOTES FOR THE USE OF PLT** before use.

- Load the PLT into a dispenser that fits the PLT
- Remove the PLT cap.

7-2. L-SYRINGE (High, Medium and Super Low)


- Hold the nozzle and turn the cap counterclockwise for removal. [1]
- Hold the nozzle and place the Dispensing Tip (metal tip) onto it. Turn the Dispensing Tip clockwise until it is securely locked. [2]

- Accessories and Dispensing Tip are exclusively designed for the L-SYRINGE of ESTELITE UNIVERSAL FLOW. Use the attached Dispensing Tips (metal tips).
- Dispensing Tips (TOKUYAMA Dispensing Tip) are also sold separately.
- Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

7-3 SYRINGE (Medium)

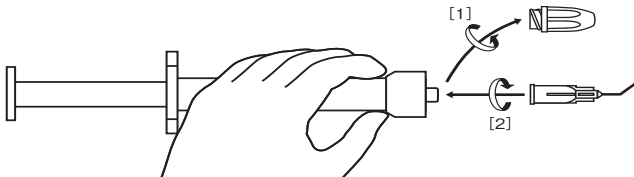- Hold the nozzle and turn the cap counterclockwise for removal. [1]
- Hold the nozzle and place the Dispensing Tip (plastic tip) onto it. Turn the Dispensing Tip clockwise until it is securely locked. [2]

- Accessories and Dispensing Tip are exclusively designed for the SYRINGE of ESTELITE UNIVERSAL FLOW. Use the attached Dispensing Tips (plastic tips).
- Dispensing Tips (TOKUYAMA Plastic Tip) are also sold separately.
- Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

8. Filling and contouring

Cavities can be directly filled from the Dispensing Tip/PLT, or indirectly filled with an instrument after extruding the paste on the mixing pad. Increments should not exceed the indicated curing depth at a time (please refer to the aforementioned table).

- DO NOT apply excessive force to extrude the material from the syringe or PLT immediately after removal from the refrigerator.
- DO NOT mix ESTELITE UNIVERSAL FLOW with other brands of resin composite or other shades of ESTELITE UNIVERSAL FLOW to avoid incomplete cure or entrapment of air bubbles.
- In case of syringe, to avoid cross infection, after extruding the paste, remove and discard the Dispensing Tip, wipe the nozzle with gauze and replace the cap immediately.
- This product is provided non-sterile and is not intended to be sterilized by the user.

9. Curing

Light-cure each increment for at least the indicated time (please see aforementioned table).

- If other brands of composite resins are layered over the cured composite, follow their instructions.

10. Finishing

Shape and polish the restoration. For finishing, use finishing discs and/or fine finishing diamond points. Use metal finishing strips or vinyl polishing strips for proximal surfaces. For polishing, polish with rubber points or any suitable polishing tools. For final polishing, use felt discs or cotton wheels with polishing paste, or suitable polishing tools.

- Instruct the patient adequately on the maintenance of the restoration after treatment.

The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the dental professional to ensure the product is suitable for application before use. Specifications are subject to change without notice. When the product specification changes, the instructions and precautions may change also.

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Packaging waste disposal guide

(According to European Commission Decision 97/129/EC)

ENGLISH

Packaging waste disposal guide

| | |
|---------------------------|--|
| PLT (cap and body) | Dispose of as industrial waste. |
| SYRINGE (cap and body) | Dispose of as industrial waste. |
| Packaging bag for PLT | PP5 (Polypropylene): Recycling. Follow local instructions for disposal. |
| Packaging bag for tip | LDPE4 (Low density polyethylene): Recycling. Follow local instructions for disposal. |
| Paperbox and/or partation | PAP21 (Non-corrugated fibreboard): Recycling. Follow local instructions for disposal. |
| IFU and/or guide | PAP22 (Paper): Recycling. Follow local instructions for disposal. |

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